

Lyofoam® Max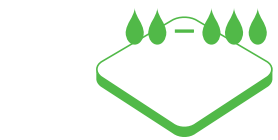

USA 1-800-882-4582
Canada 1-800-494-5134

Manufacturer
Mölnlycke Health Care AB
 Gamlestadsvägen 3C, Box 13080, SE-402 52 Göteborg, Sweden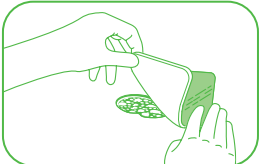

MÖLNLYCKE
HEALTH CARE

www.molnlycke.com

en Do not use if package is damaged	no Må ikke brukes hvis forpakningen er skadet
de Bei beschädigter Verp. verwerfen	sl Ne uporabite, če je ovoj poškodovan
fr Ne pas utiliser si l'emballage est endommagé	zh 如内包装破损则不可使用。
tr Ne kullanmayın	bg Не използвайте, ако опаковката е увредена
nl Niet gebr. als verpakk. beschadigd is	ro Nu utilizați dacă ambalajul e deteriorat
sv Skall inte användas om förpackningen är skadad	sk Nepoužívaťe, ak je obal poškodený
it Non utilizzare se la confezione è danneggiata	tr Ambalaj hasarlıysa kullanmayın
fi Älä käyttää jos pakkaus vaurioitunut	lt Nenaudoti, jei pakuoūtė pažeista
pl Należy uważać, aby opakowanie nie było uszkodzone	lv Nelietot, ja iepakojums ir bojāts
da Brug ikke, hvis pakken er beskadiget	el Kahjustunut, ei pakkausta käytettäväksi
el Μην το χρησιμοποιείτε αν η συσκευασία είναι καταστραμμένη	ru Не использовать, если упаковка повреждена
pl Nie stosować, jeśli opakowanie jest uszkodzone	hr Ne koristiti ako je pakiranje oštećeno
cs Nepoužívejte, je-li obal poškozen	sr Ne koristiti ako je omot oštećen
hu Ne használja, ha a csomagolás sérült	ar لا تستعمل إذا كان غلاف التغليف الداخلي تالفًا

en Lyofoam Max

Absorbent foam dressing

Product description
Lyofoam Max is a highly absorbent dressing. It consists of polyurethane foam and a waterproof backing film with high permeability to allow effective vapour transfer.

Mode of action
Lyofoam Max is a soft foam dressing that absorbs and retains exudate and maintains a moist wound environment.

Intended use
Lyofoam Max is designed for the management of moderate to highly exuding wounds.

Instructions for use
1. Cleanse the wound in accordance with normal procedures.
2. Select a suitable size of Lyofoam Max so that the dressing overlaps the wound margins by approximately 2cm/0.8in. Cut the dressing to size if necessary.
3. Place the foam side directly to the wound surface.
4. Secure the dressing with an appropriate bandage or with tape. The tape should only be applied to the edge of the dressing.

In case of venous leg ulcers compression therapy may be used in conjunction with Lyofoam Max.

Frequency of change
Lyofoam Max may be left in place for several days depending on

the nature and condition of the wound and the rate of exudate production. More frequent dressing changes may be required at the start of treatment.

Precautions
• In case of signs of infection (e.g. excessive pain, swelling), consult a health care professional for adequate infection treatment.
• Do not cover Lyofoam Max with films or tapes, as this will impair the effectiveness of the dressings.
• Sterile. Do not use if inner package is damaged or opened prior to use. Do not re-sterilise.
• Do not reuse. If reused performance of the product may deteriorate, cross contamination may occur.
• Do not use Lyofoam Max together with oxidising agents such as hypochlorite solutions or hydrogen peroxide.
• Do not use after expiry date. If the product is used after the expiry date product properties cannot be ensured.
• For external use only.

Other information
At first sloughy wounds may exude profusely and appear to worsen. This is a normal part of the cleansing process and should not cause concern provided there are no clinical signs of infection or deterioration of the wound present.

Storage
Lyofoam Max should be stored in dry conditions below 25°C (77°F)

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Mölnlycke Health Care, the word mark and logo symbol are trademarks of Mölnlycke Health Care AB