Tubifast[®]

WITH

2-WAY STRETCH[®]

TECHNOLOGY

ELASTICATED VISCOE TUBULAR BANDAGE

Tubifast 2-Way Stretch

Using Tubifast 2-Way Stretch for dressing retention

- Select the correct size of Tubifast 2-Way Stretch and cut to size if necessary.
- Stretch the Tubifast 2-Way Stretch over the hands and cover the dressing.
- Tubifast 2-Way Stretch may be applied in a double layer for extra security.

Using Tubifast 2-Way Stretch for patch wrapping

- Select the correct size of Tubifast 2-Way Stretch to cover the affected area and cut to size, if necessary.
- Bathe the affected area and pat dry.
- Where indicated by a health professional, use an appropriate topical preparation on badly affected areas.
- Apply a suitable emollient to the skin, using a downward motion.
- Apply Tubifast 2-Way Stretch to the affected area.
- If wet wrapping, apply two layers of Tubifast 2-Way Stretch. Soak the first layer in clean warm water, squeeze out any excess water and apply whilst still warm and damp. A dry Tubifast 2-Way Stretch can then be applied as a top layer.

- As an alternative to wet wrapping, Tubifast 2-Way Stretch can be applied as a single dry layer over emollients to help keep the emollients in place for longer.

Tubifast 2-Way Stretch is available in a range of widths. Please see the carton for further information.

If soiled with bodily fluids, discard appropriately and replace.

Precautions

If wet or dry wrapping: discontinue use and seek immediate medical advice if condition worsens.

If this item is intended for use as part of the wet wrapping technique, full written application instructions are available from Mölnlycke Health Care, Box 13080, SE-402 52 Göteborg, Sweden. This product and the packaging does not contain Natural Rubber Latex. Avoid naked flames and ignition sources, especially when used in conjunction with paraffin based products.

Fibre content: Viscose, elastane, polyamide.

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